



Management of Hemothorax

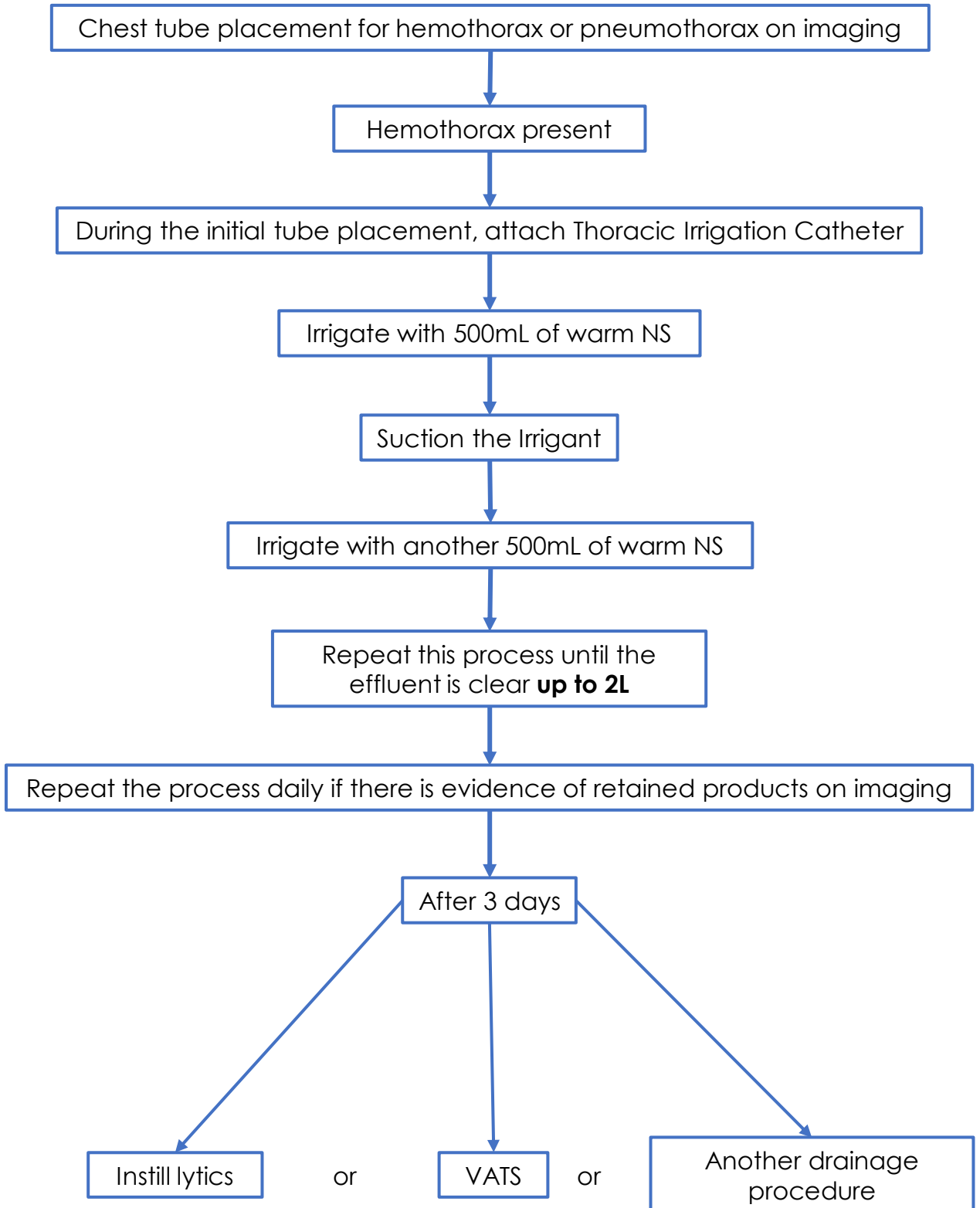

Management of Hemothorax


1

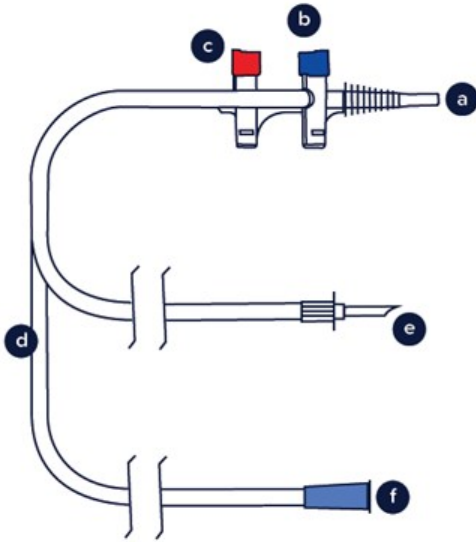

Check the Probe(s) for Damage

Dispose of probe if damaged. **a** Nozzle, **b** Irrigation Trumpet Button (blue), **c** Suction Trumpet Button (red), **d** Tubing, **e** Bag Spike with Tip Protector, **f** Suction Connector.


2

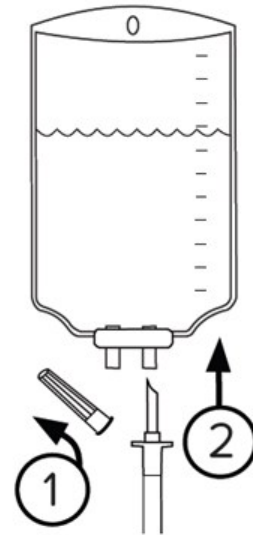

Attach to Irrigation Fluid

Remove the Tip Protector from the Bag Spike (step 1), and insert the Bag Spike into the lavage fluid (e.g. saline) bag (step 2). Depress the blue Irrigation Button to gravity-prime the unit until fluid exits the Nozzle and all air has been purged.


3

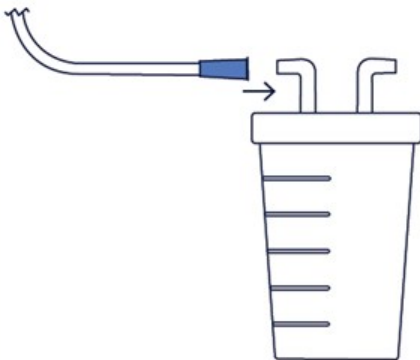

Attach to Suction

Attach the Suction Connector to the inlet fitting of the suction canister. Depress the red Suction Button to check suction.

4

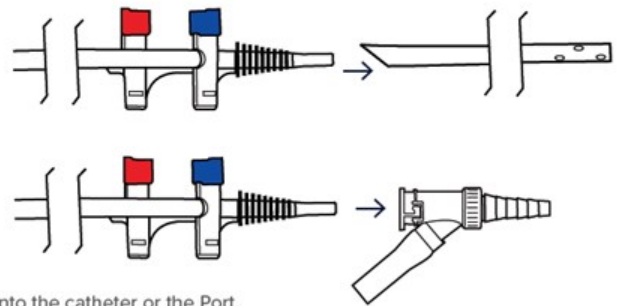


Insert Nozzle

Insert the Nozzle into the catheter or the Port.

5

Activate Irrigation or Suction

To activate irrigation, depress the blue Irrigation Button. To activate suction, depress the red Suction Button.

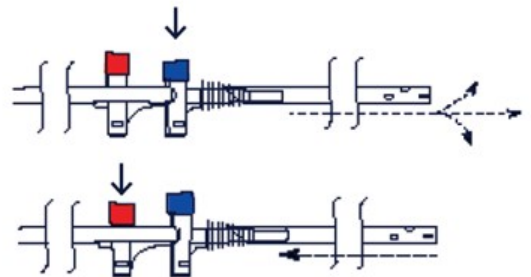


Videos for the Management of Hemothorax (Please click on the Photos to watch the videos)



Product Overview & Procedure



Procedure Only-
(included at the end of the
previous video)

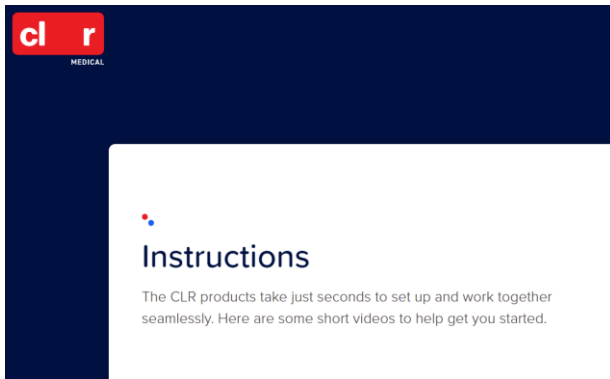

Multiple product demonstrations



**Demonstration of Irrigation
tube in use**